



MEDUMAT Standard² Using the PC Simulator

The PC simulator for MEDUMAT Standard² is intended as a supplemental training tool for all users of MEDUMAT Standard².

It can be used with the following operating systems: Windows XP, Windows Vista, Windows 7 and Windows 8.

The language of the simulation is selected automatically according to the language settings of the computer.


The operating philosophy of the simulator is the same as that for the device:


- **Select function:** Click any function key or navigation knob.
- **Adjust setting:** Click the left or right side of the displayed navigation knob or the arrow key in order to change a setting or to navigate the menus.
- **Confirm selection:** Click the middle of the navigation knob or the ENTER key.

The most **important operating steps** are summarized below:

1. Start Simulator

1. Start the **EXE**-file.
2. The simulator automatically starts (no lead time).

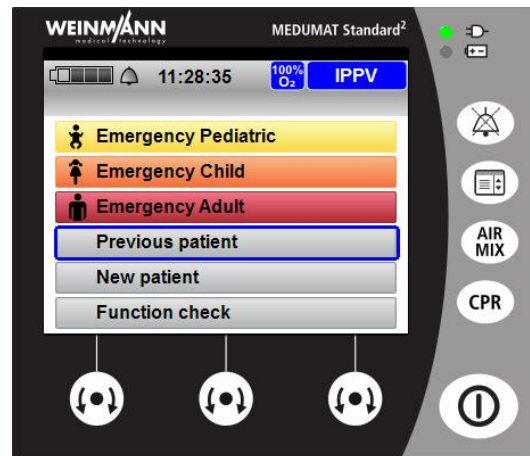


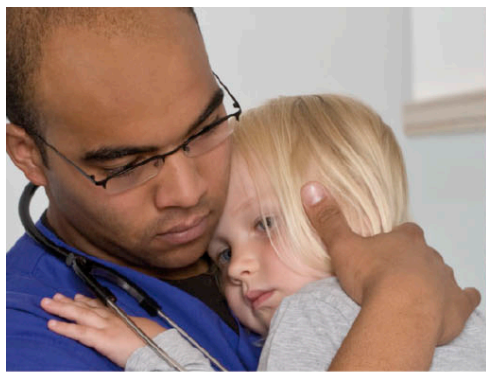

MEDUMAT Standard² Using the PC Simulator

2. Save Screenshot

1. Start the simulator.
2. With the right mouse button click the simulator monitor.
3. Click the button "save screenshot".
4. Save the screenshot anywhere you like.

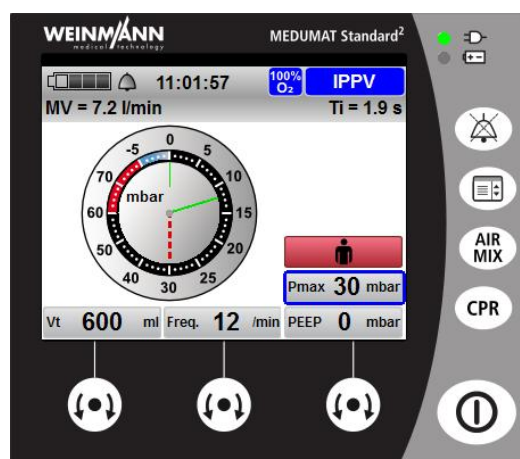

Note: Service mode cannot be activated in the simulator.