

Instruction for unpacking ZIP-files to perform software updates

Dear Sir or Madam,

To make WEINMANN medical devices even more suitable for you and your patients, we have made some improvements to the device software. To ensure a smooth progression of the software procedure, we compiled a detailed instruction with all software procedure steps.

To perform a software update, please proceed as follows:

1. Connect your storage medium to your computer.
2. Copy the current software (*.zip) onto the storage medium.
3. Open the directory of your storage medium.

Note:

The update file is in a ZIP file format on your storage medium. Several related files are compiled and compressed for storage in a ZIP file.

If you have a program to unpack ZIP files, such as *7-Zip*, *WinRAR* or *WinZip*, continue with Step 4. Otherwise, go on to Step 7.

4. To extract/unpack the contents of the ZIP file, right-click the compressed file to open the context menu.
5. From the context menu select "Extract all".
6. After it has been extracted/unpacked, the update file (recognizable by the ending *.hex) is at the top level of the storage medium. Continue now with Step 12.
7. Right-click the ZIP file to be unpacked.
8. Select "Extract all" from the sub-menu.
9. Then select the storage medium as the destination.
10. With a double-click open the newly created folder on the storage medium.
11. Drag the update file (recognizable by the ending *.hex) to the top level of your storage medium.
12. Switch on the device to be updated.
13. Insert the storage medium into the port on your device.
14. Follow the steps to make an update as described in the Instructions for use. The current version of the Instructions for use can be found at www.weinmann-emergency.de

Do you have any questions? For additional information please contact your WEINMANN Emergency reference sales person or contact our after sales service at 040-881896-122 or AfterSalesService@weinmann-emt.de.

Kind regards,

Your WEINMANN Emergency Team